

## **Directions for use:**

Shake container before each use. Dilute 1/8 to 1/4 oz. of **Instra-Ease II™** concentrate to 1 gallon of water, in a tray (preferably stainless steel). Soak instruments for 30 seconds, remove and let drain. Follow up with sterilization. Diluted lubricant bath will last up to 14 days depending upon water quality. Bath may need replaced more often depending upon volume of instrument processed.

Keep air away from bath by covering tray when not in use.

**Instra-Ease II™** Instrument Lubricant is not a sterilant. Instruments must be sterilized following treatment with this product.